


LIGAMENTOTAXOR[®]

Clinical case, Prof. Ph. Pélissier / Bordeaux CHU, France

Male - 32 years

Fracture DIP 4th finger, left hand.

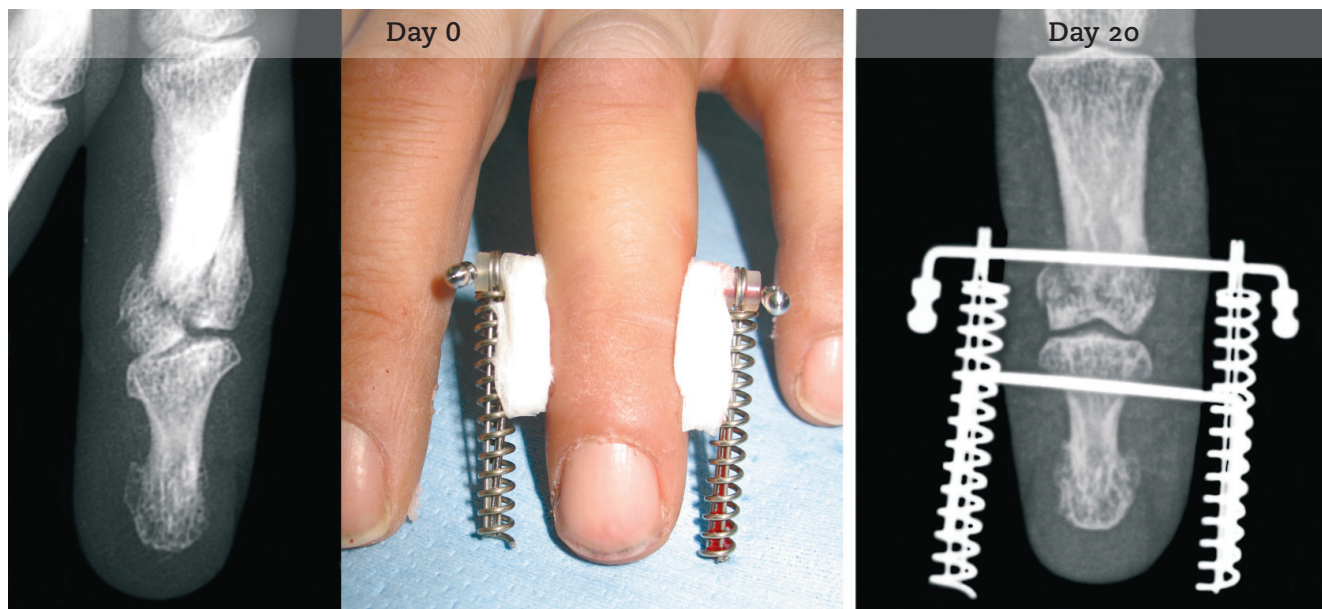

Post operative

Rehabilitation protocol:

- D0 - D10 Articular traction, no mobilisation.
- D10 - D25 Passive mobilisation.
- D25 - D40 Active mobilisation.
- D40 Apparatus is removed.

Results after 2 months:




LIGAMENTOTAXOR®

Clinical case, Prof. Ph. Pélissier / Bordeaux CHU, France

Male - 54 years

Crush fracture of proximal interphalangeal (PIP) joints, 3rd and 4th fingers.

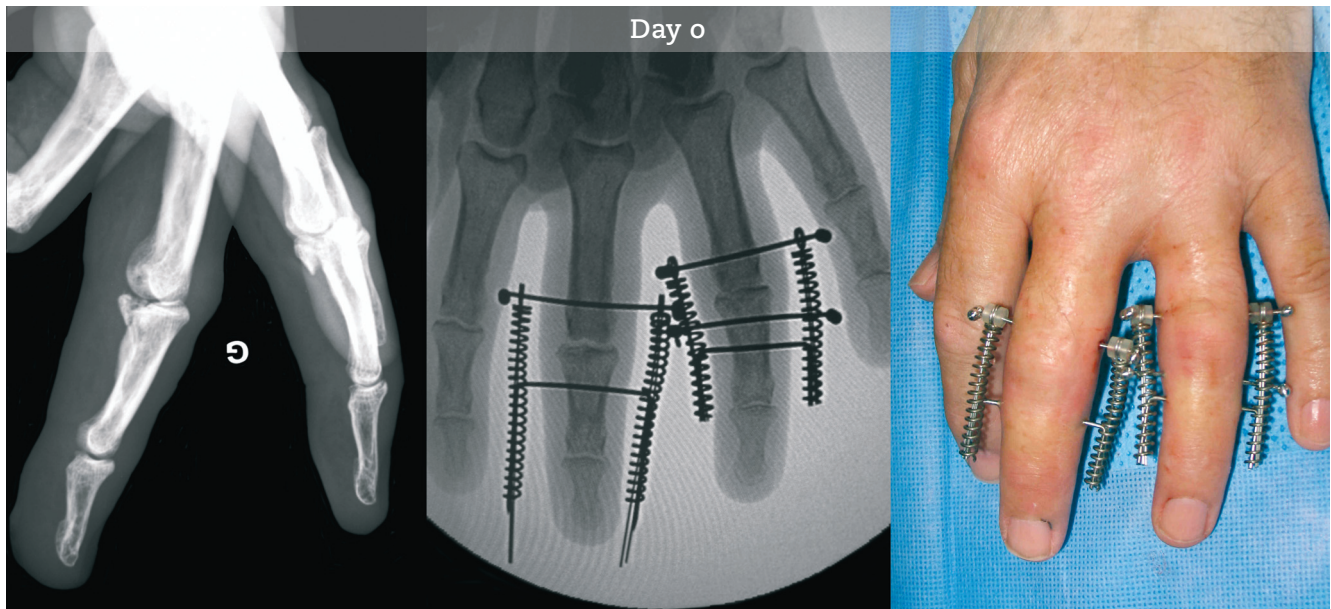

On 3rd finger, articular traction with LIGAMENTOTAXOR® to reduce fracture.

On 4th finger, initial posterior displacement at the base of P2 is reduced by a supplementary transverse pin supported by lateral bars.

Post operative

Rehabilitation protocol:

- D00 - D10 Articular traction, no mobilisation.
- D10 - D25 Passive mobilisation.
- D25 - D40 Active mobilisation.
- D40 Apparatus is removed.

Results after 4 months:



Video at 18 months





www.arex.fr/QRc/PELISSIER-IPP.mp4

LIGAMENTOTAXOR®

Clinical case, Prof. M. Merle / Kirchbierg Hospital, Luxembourg

Male - 18 years

Fracture at base of index P2, left hand.

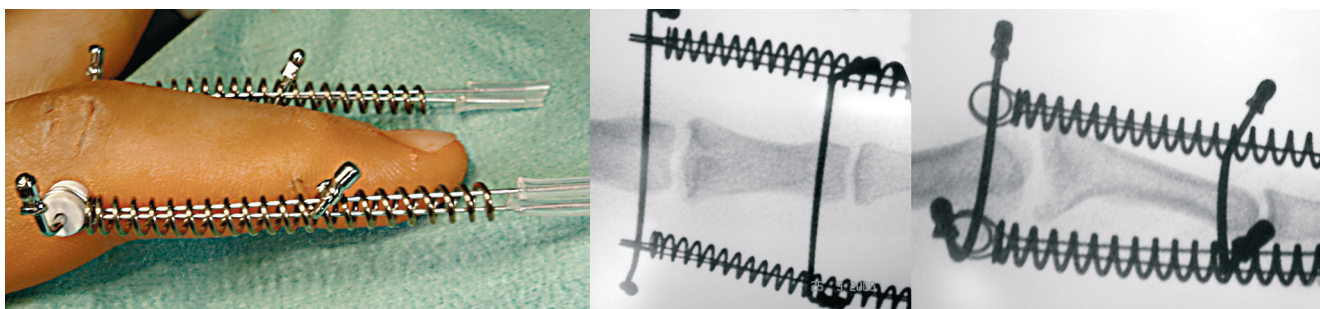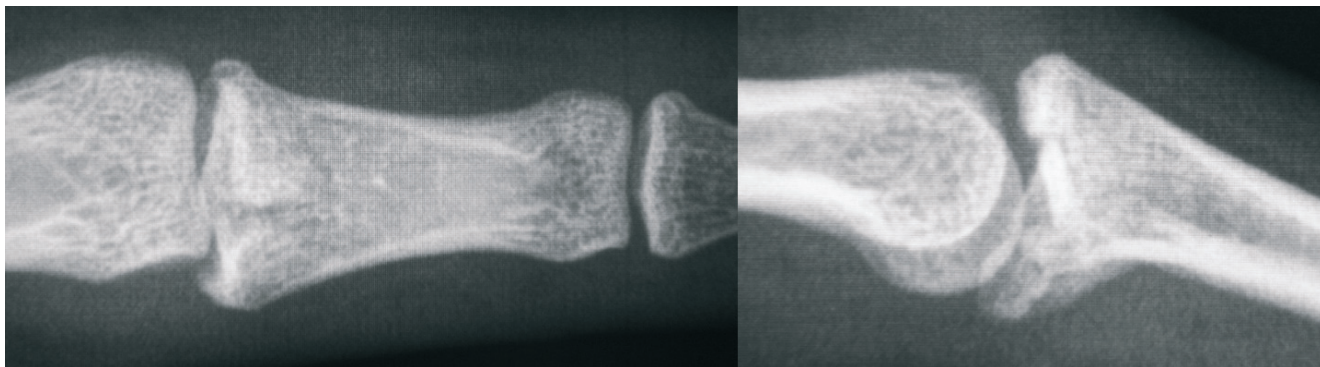

The proximal pin must be positioned in strict alignment with the PIP joint axis of rotation.
Distraction is adjusted to achieve optimal remodelling of base of P2.

Post operative



Video at 2 months:



www.arex.fr/QRc/MERLE-P2.mp4


LIGAMENTOTAXOR®

Clinical case, Prof. Ph. Liverneux


SOS MAIN Strasbourg Hôpitaux Universitaires, France

Male - 47 years

Fracture at base of first phalanx of thumb, right hand.

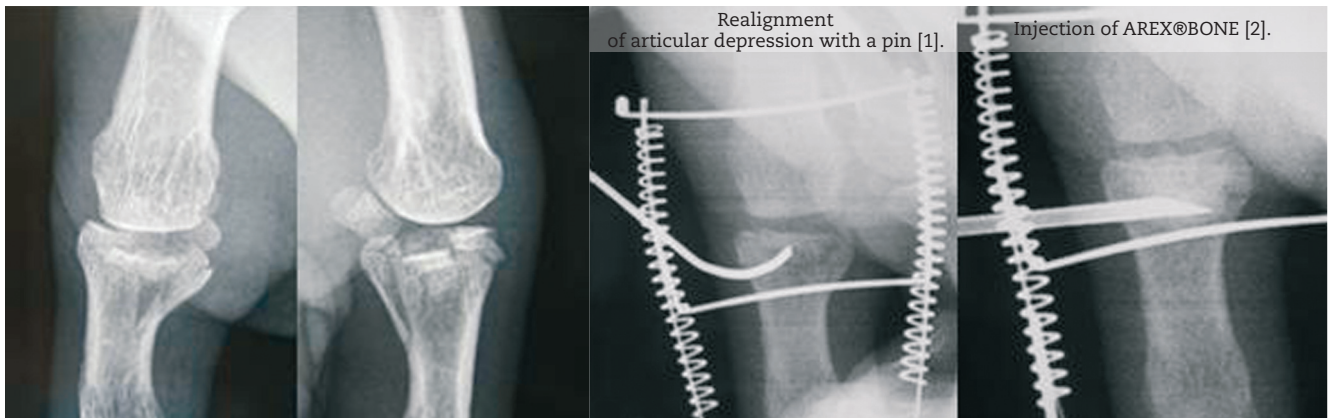

D0: Application of LIGAMENTOTAXOR®. [1] & [2] percutaneous procedure throughout.

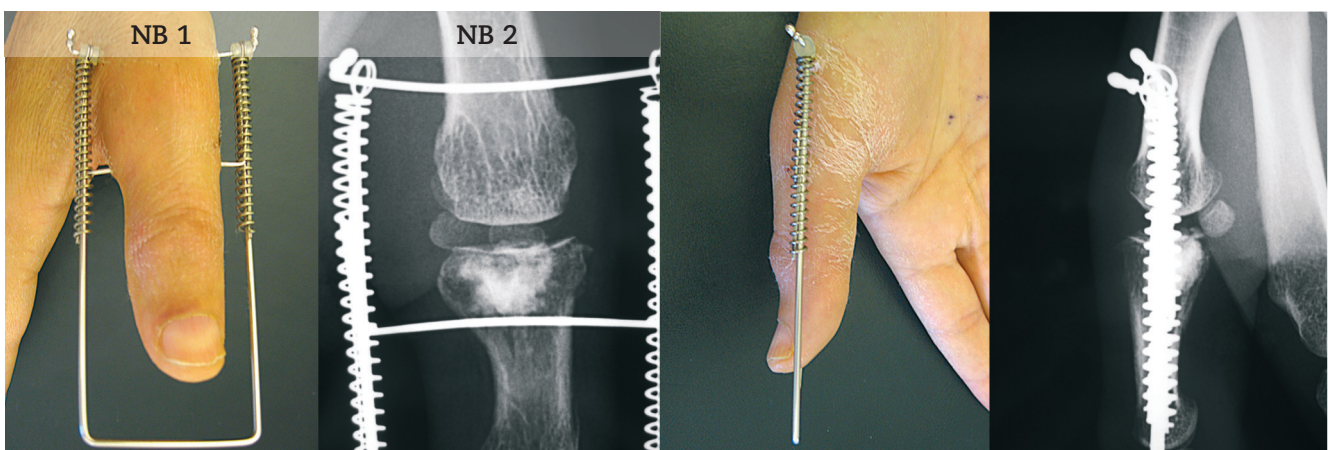

NB 1: 1.5 mm Ø Kwire will be bent "U" shape in order to realise a stabilizing frame.

NB 2: MP joint is under strong tension, Kwires and springs are curved.

Post operative

D1 The patient is able to use his hand and even write:

D30 Removal of splint.



www.arem.fr/QRc/LIVERNEUX-15D.mp4

At 5 months, excellent results:

- No pain.
- Mobility and strength identical to contralateral hand.



LIGAMENTOTAXOR®


Clinical case, M. Schoofs, MD / Lille, France

Woman - 44 years

Dorsal wound to left hand caused by lawnmower blade, 4th digit involvement.

Fracture of 4th metacarpal and 1st phalanx.

Comminuted multifragmentary fracture and extensor dilaceration extending to PIP joint, including metacarpo-phalangeal capsule.

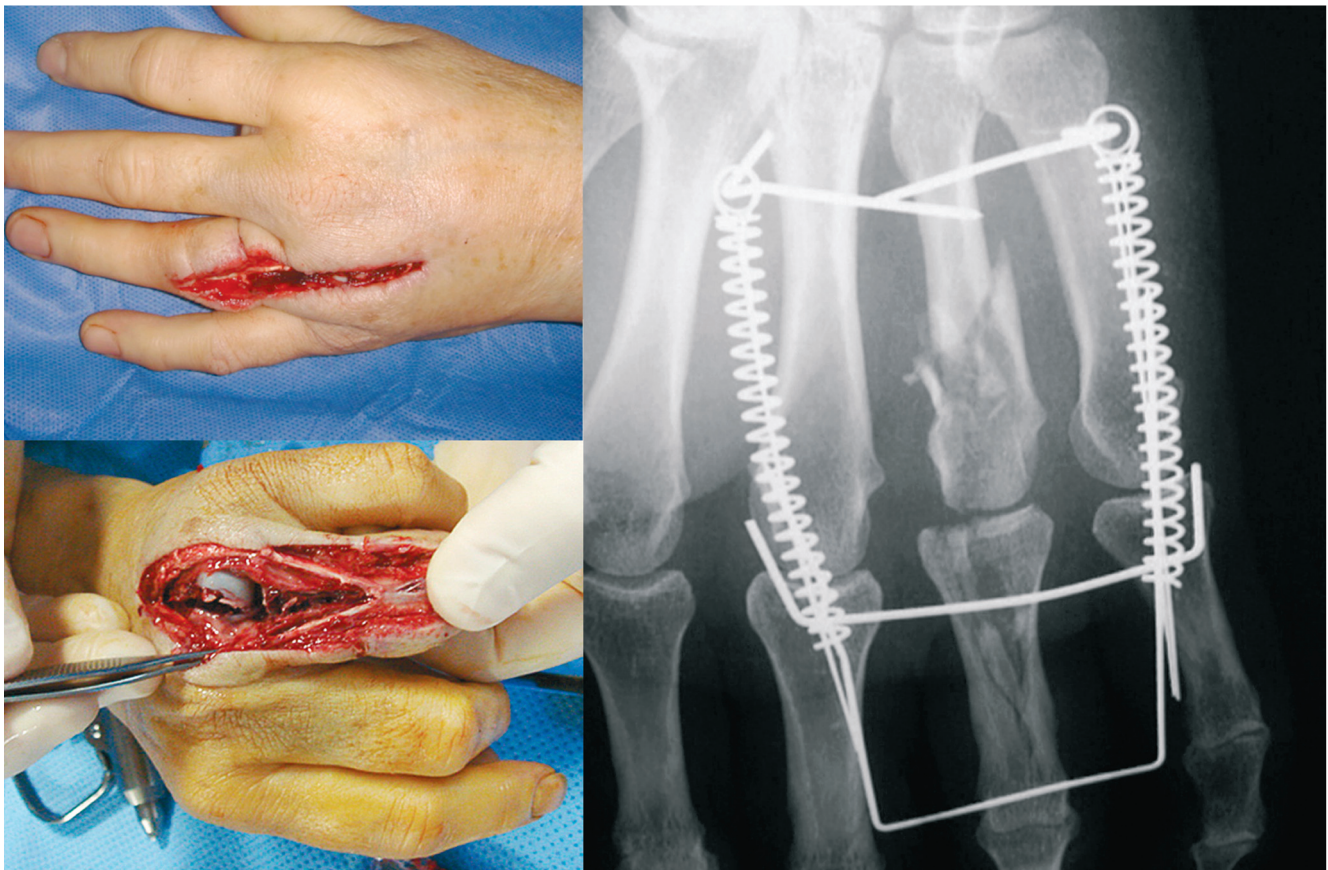

6 months



www.arex.fr/QRc/SCHOOFs-CC2.mp4

LIGAMENTOTAXOR[®]


Clinical case, M. Schoofs, MD / Lille, France

Male - 30 years

Irreducible, one month old luxation of PIP joint, 5th digit of right hand

Seen September 9, 2009. Operated on September 12, 2009 with placement of LIGAMENTOTAXOR[®].

Removed on November 4, 2009.

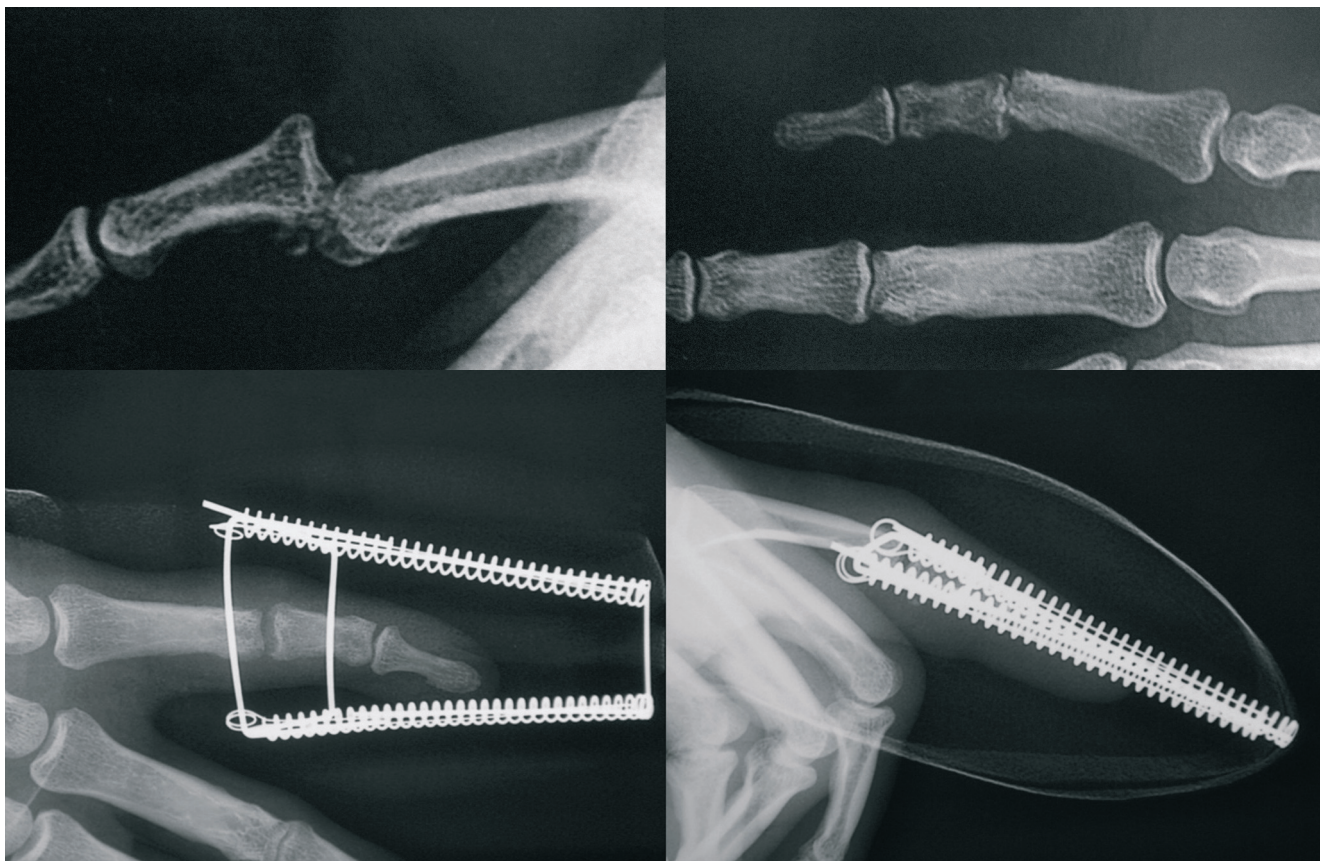

2 months



www.arex.fr/QRc/SCHOOFs-CC3.mp4

Cr ation : e-moneaux / Ref. 114 ed.05 -   AREX  - 2013


LIGAMENTOTAXOR®

Clinical case, M. Schoofs, MD / Lille, France

Male - 35 years

With open crush injury to phalanx P2 and severe cutaneous contusion and tissue loss.
No severance of flexor tendon.

Treatment: resection, partial suture, placement of LIGAMENTOTAXOR®
and managed wound care.

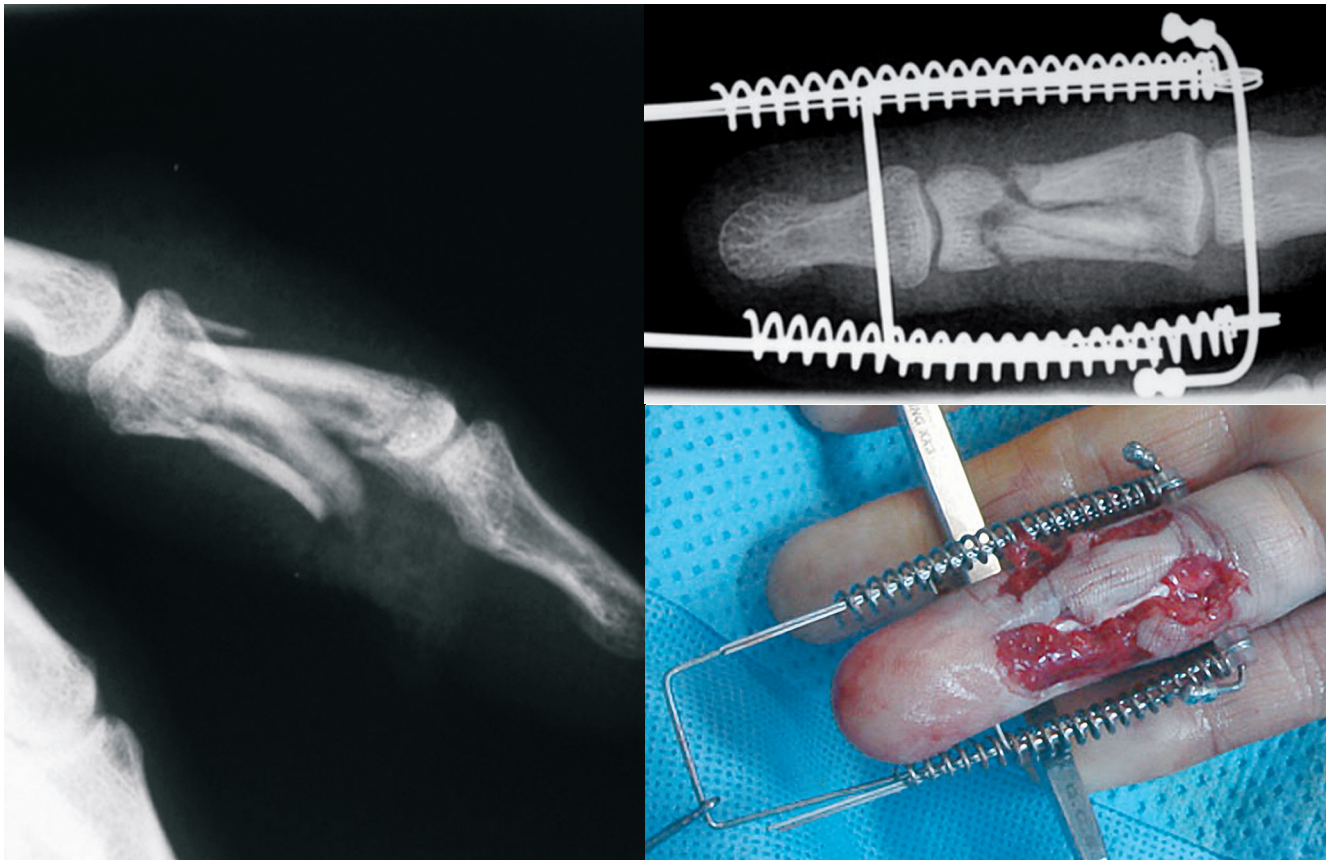

8 weeks, post initial traumatic event



www.arex.fr/QRc/SCHOOFs-CC1.mp4

Création : e-moncaux / Ref. 114 ed.05 - © AREX® - 2013


LIGAMENTOTAXOR®

Clinical case, H.-G. Damert, MD / University Hospital, Magdeburg, Germany

Male - 25 years

Repetitive traumatic injuries to fifth finger — typical goalkeeper fractures of proximal and middle phalanges.

Distal phalanx fracture sustained in the past.

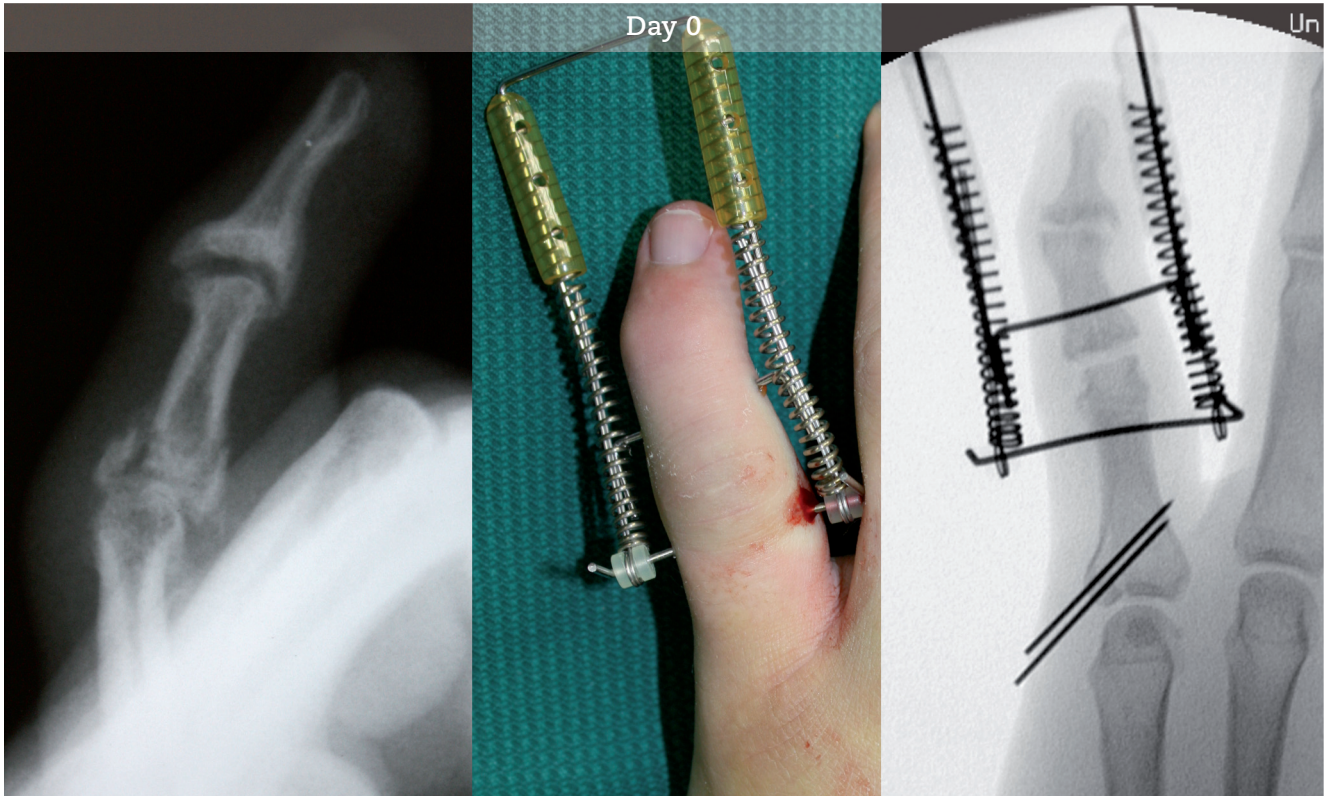

Post operative

Rehabilitation protocol:

- D0 - D10 Articular traction.
- D10 - D20 Passive mobilisation.
- D20 - D42 Active mobilisation.
- D42 LIGAMENTOTAXOR® is removed.

