


FORGETMENOT® - ref.: FMNB (blue) and FMNY (yellow)

Simple answer to a real issue*: digit tourniquet **ForgetMeNot®**.

- available in yellow or blue
- reusable
- sold in packs of 10
- sold no sterile

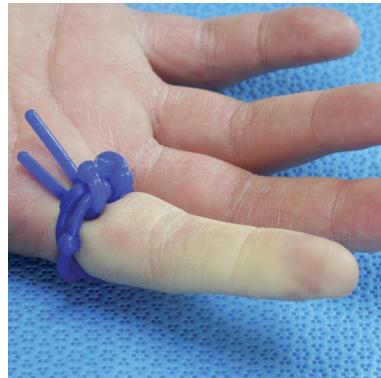

* Résultats fonctionnels à long terme d'ischémies digitales sous garrot : à propos de 2 observations.
SOFLOT 2010 / Florence Mallard (Angers) Julien Richou, Dominique Le Nen

DISTRIBUTED BY




www.arex.fr

FORGETMENOT® - ref.: FMNB (blue) and FMNY (yellow)

The only specifically designed-for-purpose, reusable finger tourniquet.

- reusable
- one size-fits-all
- achieving a totally exsanguinated finger
- bright blue or yellow highly visible tails make it totally unforgettable
- removable without rolling it over the wound or any risk of cutting skin
- environmentally friendly since it is reusable

For optimal FMN® use, follow the detailed instructions below.
Placement of the FMN tourniquet is simple, easy and repeatable.
It is used under local (LA) or locoregional (LRA) anaesthesia.

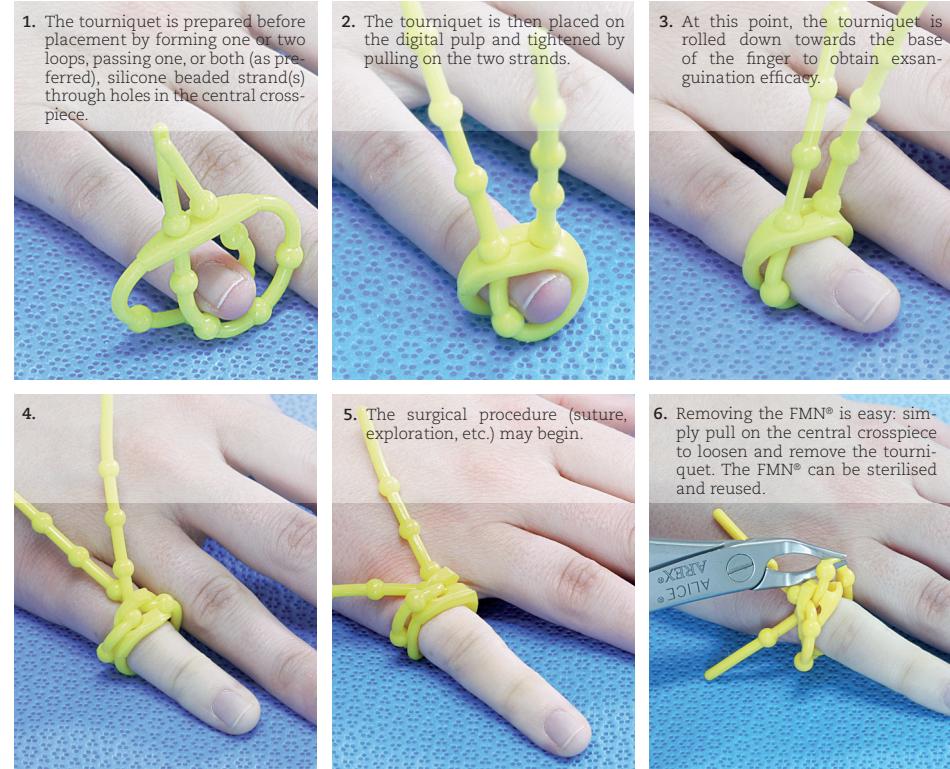

When a totally bloodless field is not required, the FMN® can be used with a single loop: the arm is held above heart level for a few minutes and the FMN® is applied directly to the base of the finger.